TorqMax[®]
Sheath Grip Accessory

INSTRUCTIONS FOR USE

Spectranetics[®]

THIS PAGE LEFT INTENTIONALLY BLANK

TorqMax® Sheath Grip Accessory

INSTRUCTIONS FOR USE

Table of Contents

Pg	Language		Instructions For Use
4	English	English	Instructions For Use
7	Bulgarian	български език	Указания за употреба
10	Croatian	hrvatski	Upute za uporabu
13	Czech	Česky	Návod k použití
16	Danish	Dansk	Brugsanvisning
19	Dutch	Nederlands	Gebruiksaanwijzing
22	Estonian	Eesti	Kasutusjuhis
25	Finnish	Suomi	Käyttöohjeet
28	French	Française	Mode d'emploi
31	German	Deutsch	Gebrauchsanweisung
34	Greek	Ελληνικά	Οδηγίες χρήσης
37	Hungarian	Magyar	Használati útmutatóaz
40	Icelandic	Íslenska	Leiðbeiningar um notkun
43	Italian	Italiano	Istruzioni per l'uso
46	Latvian	latviešu valoda	Lietošanas pamācība
49	Lithuanian	Lietuvių kalba	Naudojimo instrukcija
52	Norwegian	Norsk bokmål	Bruksanvisning
55	Polish	Polski	Instrukcja obsługi
58	Portuguese	Português	Instruções para Utilização
61	Romanian	Română	Instrucțiuni de utilizare
64	Russian	Русский язык	Инструкции по применению
67	Serbian	српски језик	Uputstvo za upotrebu
70	Slovak	Slovenčina	Návod na použitie
73	Slovenian	Slovenščina	Navodila za uporabo
76	Spanish	Español	Instrucciones de uso
79	Swedish	Svenska	Bruksanvisning
82	Turkish	Türkçe	Kullanım Talimatları

TorqMax® Sheath Grip Accessory

INSTRUCTIONS FOR USE

Table of Contents

1. DESCRIPTION.....	4
2. INDICATIONS FOR USE	5
3. CONTRAINDICATIONS	5
4. WARNINGS	5
5. PRECAUTIONS	5
6. ADVERSE EVENTS	5
7. INSTRUCTIONS FOR USE	5
8. MANUFACTURER'S LIMITED WARRANTY.....	6

Federal law restricts this device to sale by or on the order of a physician.

1. DESCRIPTION

The TorqMax® Sheath Grip Accessory is an ergonomic hand grip device used to assist in the handling and manipulation of outer support sheaths, dilator sheaths, and Spectranetics laser sheaths. The TorqMax® Sheath Grip Accessory is loaded onto a sheath from the side and remains in place on appropriately sized sheaths. Grip traction on the sheath for delivery of torque or axial force is achieved when the user squeezes the main body of the accessory. The soft inner liner of the TorqMax® device protects the sheath from damage while providing a high traction surface. The TorqMax® device may be easily relocated along the length of the sheath or easily removed during a procedure as needed by squeezing the logo-decorated finger tabs located on the side of the device (see Figure 1).

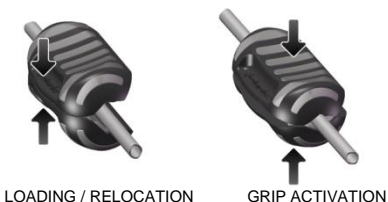
Figure 1: TorqMax® Sheath Grip Accessory

Use with other devices

The TorqMax® Sheath Grip Accessory may be used with size-compatible outer support sheaths, dilator sheaths, and Spectranetics laser sheaths. Table 1 provides specifications for use in determining TorqMax® compatibility with other devices.

Be sure to closely follow the "Instructions for Use" for each device used.

Table 1: TorqMax® Sheath Grip Accessory Size Compatibility Specifications

Minimum Sheath Outer Diameter	4.0 mm / 0.155 in / 11.9 F
Maximum Sheath Outer Diameter	7.5 mm / 0.296 in / 22.5 F

2. INDICATIONS FOR USE

The TorqMax® Sheath Grip Accessory is intended for use in providing ergonomic grip on outer support sheaths, dilator sheaths, and Spectranetics laser sheaths.

3. CONTRAINDICATIONS

None known.

4. WARNINGS

Observe all warnings for the sheath to be used as indicated in the associated "Instructions for Use".

5. PRECAUTIONS

For single use only. The TorqMax® Sheath Grip Accessory must not be resterilized and/or reused.

DO NOT resterilize or reuse this device, as these actions can compromise device performance or increase the risk of cross-contamination due to inappropriate reprocessing.

Reuse of this single use device could lead to serious patient injury or death and voids manufacturer warranties.

Do not alter the TorqMax® Sheath Grip Accessory from its original state prior to use.

Observe all precautions for the sheath to be used as indicated in the associated "Instructions for Use".

6. ADVERSE EVENTS

Refer to the "Instructions for Use" of the sheath to be used for adverse event information.

7. INSTRUCTIONS FOR USE

7.1 Sterilization

For single use only. Do not re-sterilize and/or reuse. The TorqMax® Sheath Grip Accessory is supplied sterile and non-pyrogenic. Sterility is guaranteed only if the packaged is unopened and undamaged.

7.2 Device / Packaging Care

Store devices in a dry cool place (below 60° C / 140° F) until use.

7.3 Inspection Prior to Use

Prior to use, visually inspect the sterile package to ensure that seals have not been broken. The device should be examined carefully for defects or damage. Do not use if it has apparent defects or is damaged.

7.4 Procedure Setup

Follow all recommended procedure setup steps for the sheath to be used as indicated in the associated "Instructions for Use".

7.5 Clinical Technique

1. Load the TorqMax® Sheath Grip Accessory on the proximal portion of the sheath in use and adjust its position more distal as needed. The logo-decorated finger tabs located on the side of the TorqMax® Sheath Grip Accessory can be squeezed to facilitate loading and repositioning of the accessory.
2. Squeeze the main body of the TorqMax® Sheath Grip Accessory to apply force for rotation or advancement/retraction of the sheath in use.

8. MANUFACTURER'S LIMITED WARRANTY

Manufacturer warrants that the TorqMax® is free from defects in material and workmanship when used by the stated "Use By" date and when package is unopened and undamaged immediately before use. Manufacturer's liability under this warranty is limited to replacement or refund of the purchase price of any defective TorqMax®. Manufacturer will not be liable for any incidental, special, or consequential damages resulting from use of the TorqMax®. Damage to the TorqMax® caused by misuse, alteration, improper storage or handling, or any other failure to follow these Instructions for Use will void this limited warranty. **THIS LIMITED WARRANTY IS EXPRESSLY IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.** No person or entity, including any authorized representative or reseller of Manufacturer, has the authority to extend or expand this limited warranty and any purported attempt to do so will not be enforceable against Manufacturer.

	QTY	MIN \varnothing	MAX \varnothing	NON-PYROGENIC
Symbol Meaning	Quantity	Minimum Outer Diameter	Maximum Outer Diameter	Non-pyrogenic

International Post-Market Support Line:
+31 33 434 7050
complaints@spectranetics.com

US Post-Market Support Line:
888-341-0035
complaints@spectranetics.com

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©2011 Spectranetics Corporation
Spectranetics Corporation
Spectranetics International B.V.