Instructions for Use

Spectranetics®

Instructions for Use

Table of Contents

Pg	Language		Instructions For Use
3	English	English	Instructions For Use
5	Bulgarian	български език	Указания за употреба
7	Croatian	hrvatski	Upute za uporabu
9	Czech	Česky	Návod k použití
11	Danish	Dansk	Brugsanvisning
13	Dutch	Nederlands	Gebruiksaanwijzing
15	Estonian	Eesti	Kasutusjuhised
17	Finnish	Suomi	Käyttöohjeet
19	French	Française	Mode d'emploi
21	German	Deutsch	Gebrauchsanweisung
23	Greek	Ελληνικά	Οδηγίες χρήσης
25	Hungarian	Magyar	Használati útmutató
27	Icelandic	Íslenska	Leiðbeiningar um notkun
29	Italian	Italiano	Istruzioni per l'uso
31	Latvian	latviešu valoda	Lietošanas pamācība
33	Lithuanian	Lietuvių kalba	Naudojimo instrukcija
35	Norwegian	Norsk bokmål	Bruksanvisning
37	Polish	Polski	Instrukcja obsługi
39	Portuguese	Português	Instruções para Utilização
41	Romanian	Română	Instrucțiuni de utilizare
43	Russian	Русский язык	Инструкции по применению
45	Serbian	српски језик	Uputstvo za upotrebu
47	Slovak	Slovenčina	Návod na použitie
49	Slovenian	Slovenščina	Navodila za uporabo
51	Spanish	Español	Instrucciones de uso
53	Swedish	Svenska	Bruksanvisning
55	Turkish	Türkçe	Kullanım Talimatları

Instructions for Use

Table of Contents

DESCRIPTION	3
INDICATIONS	3
CONTRAINDICATIONS	3
HOW SUPPLIED	3
DIRECTIONS FOR USE	3
WARNINGS / PRECAUTIONS	3
ADVERSE EFFECTS	4
MANUFACTURER'S LIMITED WARRANTY	4
NON-STANDARD SYMBOLS	4

Federal law (USA) restricts this device to sale by or on the order of a physician

DESCRIPTION

The Spectranetics Quick-Cross® Extreme Support Catheters are intravascular catheters. These catheters are available in a variety of lengths and tip configurations. All models have 3 radiopaque markers spaced equally along the distal shaft to aid in estimating geometry within the vascular system. The distal radiopaque marker is positioned within 3 mm of the distal catheter tip. A standard female luer is placed on the proximal end of each model. The catheter is coated with a lubricious, hydrophilic coating.

INDICATIONS

Quick-Cross® Extreme Support Catheters are intended to guide and support a guidewire during access of the vasculature, allow for wire exchanges and provide a conduit for the delivery of saline solutions or diagnostic contrast agents.

CONTRAINDICATIONS

None known.

Product Features & Model Numbers Table

Model (Ref)	Working Length	Tip Shape / Type	Hydrophilic Coating Length	Maximum Guidewire O.D.	Minimum Guide I.D.	Minimum Sheath I.D.	Maximum Catheter O.D.	Marker Band Spacing
518-084	135 cm	Straight	100 cm	0.014"	5F	4F	3.2F	15 mm
518-086	150 cm	Straight		0.36 mm			0.042"	
518-088	90 cm	Straight	60 cm	0.018"	5F	4F	3.4F	15 mm
518-090	135 cm	Straight	100 cm				0.044"	
518-092	150 cm	Straight					1.12 mm	
518-076	65 cm	Straight	45 cm	0.035"	N/A	5F	4.5F	50 mm
518-078	90 cm	Straight	70 cm					
518-080	135 cm	Straight	115 cm					
518-082	150 cm	Straight	130 cm					

HOW SUPPLIED

The Spectranetics Quick-Cross® Extreme Support Catheters are supplied **STERILE**. The devices are designated and designed for **SINGLE USE ONLY** and must not be re-sterilized and/or reused.

The sterility of the product is guaranteed only if the package is unopened and undamaged. Before use, visually inspect the sterile package to ensure that the seals have not been broken. Do not use the catheter if the integrity of the package has been compromised. Do not use catheter if its "Use Before Date," found on package labeling, has been passed.

Before use, examine carefully for defects, all of the equipment to be used. Do not use any equipment if it is damaged.

After use, dispose of all equipment in accordance with applicable specific requirements relating to hospital waste, and potentially bio-hazardous materials.

DIRECTIONS FOR USE

Note: Follow instructions for use for all equipment to be used with the Quick-Cross® Extreme Support Catheter.

- Preparation: Using sterile technique, open the sterile package. Gently remove the luer hub from the card and slide the catheter out of the packaging. Fill a sterile standard luer-lock syringe with sterile saline and flush the lumen.
- Insertion: Through a previously inserted, appropriately sized guiding catheter or introducer sheath, introduce the catheter over an appropriately sized guidewire (see specifications) using standard technique.
- Advancement: Use fluoroscopic guidance when advancing the catheter to the desired location within the vasculature.
- Removal: Gently withdraw the catheter using standard technique, being careful to maintain guidewire position if the guidewire is to remain in place.

Infusion: To perform infusion, withdraw the guidewire and reference the specifications for maximum infusion pressure.

Note: Do not exceed the maximum infusion pressures.

After use, dispose of all equipment in accordance with applicable requirements relating to hospital waste, and potentially bio-hazardous materials.

WARNINGS / PRECAUTIONS

- Store in a cool, dry place. Protect from direct sunlight and high temperature (greater than 55°C or 131°F).
- Maximum Infusion Pressure: **300 psi** for 0.014" & 0.018" catheters and **500 psi** for 0.035" catheters.
- The catheter is designed and intended for one time use only. **Do not re-sterilize and/or reuse.**
- DO NOT re-sterilize or reuse this device, as these actions can compromise device performance or increase the risk of cross-contamination due to inappropriate reprocessing. Reuse of this single use device could lead to serious patient injury or death and voids manufacturer warranties.
- Do not use if device or packaging is damaged.
- Use the catheter prior to the "Use By" date (Expiration Date) specified on the package.
- The catheter should only be used by physicians qualified to perform percutaneous vascular interventions.

ADVERSE EFFECTS

Vascular catheterization and/or vascular intervention may result in complications including but not limited to:

- Vessel dissection, perforation, rupture or total occlusion
- Infection
- Hematoma
- Unstable angina
- Embolism
- Hypo/hypertension
- Acute myocardial infarction
- Arrhythmia, including ventricular fibrillation
- Death

MANUFACTURER'S LIMITED WARRANTY

Manufacturer warrants that the Quick-Cross® Extreme is free from defects in material and workmanship when used by the stated "Use By" date and when package is unopened and undamaged immediately before use. Manufacturer's liability under this warranty is limited to replacement or refund of the purchase price of any defective Quick-Cross® Extreme. Manufacturer will not be liable for any incidental, special, or consequential damages resulting from use of the Quick-Cross® Extreme. Damage to the Quick-Cross® Extreme caused by misuse, alteration, improper storage or handling, or any other failure to follow these Instructions for Use will void this limited warranty. **THIS LIMITED WARRANTY IS EXPRESSLY IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.** No person or entity, including any authorized representative or reseller of Manufacturer, has the authority to extend or expand this limited warranty and any purported attempt to do so will not be enforceable against Manufacturer.

NON-STANDARD SYMBOLS

Non-Pyrogenic	Maximum PSI/kPa
GW Compatibility	Sheath Compatibility
Distal Marker Spacing	Working Length
Maximum OD	Shape

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©2011 Spectranetics Corporation
Manufactured by Spectranetics Corporation